



# HTS

**Hygienic, simple, affordable and adaptable** to most toilets, the Hygiene and Toileting System (HTS) is designed to improve the health of the client and the life of the caregiver.

The versatility of the HTS makes it ideal for anyone dealing with tight budgets or tight spaces. Because it accommodates toileting, showering and bathing, the HTS eliminates the need to buy separate equipment for each activity.



Find letters of medical necessity  
and informative articles at:  
[www.rifton.com/hts](http://www.rifton.com/hts)

*“After* a year of unsuccessful training attempts with a particular student, we are excited to say that the first day we put him on a Rifton HTS he was so comfortable that he voided twice in the same day.”

*Kerri Wright, Special Ed Teacher  
Orange Ulster BOCES, NY*

HTS (Z122 tilt-in-space base)

DESIGNED FOR USE WITH THE  
**MOVE**<sup>®</sup>  
PROGRAM

# HTS features

**One of the most affordable** toileting systems available today, the HTS can be configured in myriad ways to fit your clients. Options range from the basic model with seat, back, base and armrests to the fully accessorized version.



## **Infinite adjustability**

Adjust the height, seat depth and backrest angle independently for optimal positioning.



**Foam molded cushions**  
Soft foam pads on the back and seat provide comfort for your clients and easy cleaning for caregivers.



**Better mobility**  
The large caster option improves mobility and has better resistance to picking up debris.



**Accessible for clients**  
A flip-up footboard makes sit-to-stand transitions easy.

# Any client, any toilet, any situation

**For clients,** each element of the HTS provides the best opportunity for hygiene. Its wide range of adjustment and varied support options help clients maintain positioning that promotes effective toileting, and the tilt-in-space feature and adjustable headrest make showering easier.

**For caregivers,** the HTS is the paragon of adaptability. It can be used on or over a round front or elongated toilet, or as a free-standing commode. It can adjust to children as they grow, and includes components and supports for clients of every size and need.





**ON**

**Placing the HTS** on any model of toilet, round front or elongated, enables the client to share the same toilet as the rest of the family. Mounting bars make for easy, secure attachment of the chair to the toilet base. Removal is just as easy, no tools required.



**OVER**

**Wheel the HTS** over toilets of nearly any size and style, enabling the client to share the same toilet as the rest of the family. Or use the stationary base on toilets up to 15" tall.



**OFF**

**Use the HTS** off the toilet as a stand-alone commode in any location.



# The chair that grows with you

The HTS comes in three sizes, each with a wide range of adjustment. As your child grows the chair can adapt, giving you years of useful service.



## Which chair is larger?

**The chairs in the photo on the right are exactly the same size:** one is adjusted to its smallest setting to accommodate a 5-year-old, the other at its largest setting fits an 11-year-old. The HTS fits a wide range of sizes, body types and special needs, allowing room for growth over time or use by multiple users.



# Easy lifts and transfers

Lifts and transfers present a major challenge to successful toileting. The HTS makes them easier with features like a gas-assisted spring tilt adjustment for sit-to-stand transfers, removable armrests for lateral transfers and a weight-bearing footboard (up to 150 lbs) for front-loading transfers.



This client's whole weight is supported by the footboard as she assists in a transfer.



The forward tilt and the easily removed armrests make sit-to-stand and lateral transfers simple.





# Tilt-in-space

**Tilt the HTS** forward (up to 5°) for easier transfer. Tilt backward (up to 25°) for showering and hair washing.



## Doubles as the perfect shower chair.

**Save space and money** with a single product for both toileting and showering. The aluminum, stainless steel and plastic construction withstands bleach and water.

The HTS tilt in space base tilts 5° forward or 25° back for showering and hair washing.

# Components

## Seat and back with seat belt



## Choice of base or mounting bar



**Mobile**  
(tilt-in-space with footboard)



**Mobile**  
(non-tilt with footboard)




**Stationary**  
(non-tilt with footboard)



**Tub base**  
(non-tilt, for medium and small HTS seats)

**Note:** Check clearance if using over a toilet.

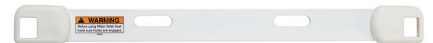

**Mounting bar** (round front toilet and large HTS)  
Round-front toilets will measure 17" or less from the centerline of the seat bolts to the front edge of the porcelain bowl.



**Mounting bar** (elongated toilet, excluding large HTS)  
Elongated toilets will measure more than 17" from the centerline of the seat bolts to the front edge of the porcelain bowl.

Use the **Mounting bar** to attach the HTS seat to round-front or elongated toilets. The easy on-and-off feature allows caregivers to quickly position or remove the seat.


## Armrests



**Armrests** come in three heights and are easily removable for lateral transfers. The height selection allows the HTS to grow with a child or accommodate the needs of multiple clients.

**Low:** 6" high x 9" long  
**Medium:** 7" high x 10½" long  
**High:** 9" high x 12" long

## Components (continued)

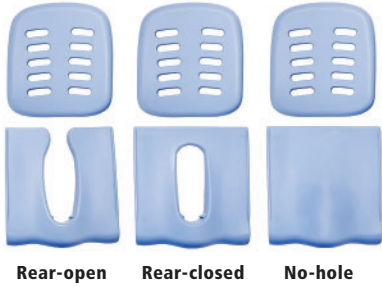

### Seat and back pads

Waterproof closed-cell foam pads are easily removed for cleaning. The unique rear-open seat pad enables hygiene care while the client is still seated and relaxed. Rear-closed and no-hole pads provide additional support. The no-hole pad comes with a rigid hole filler.



### Push handles

A set of push handles allows the caregiver to easily move a client seated in the HTS.



### Headrest

The curved headrest provides support for clients with limited head control and adjusts in height, depth and angle.



### Butterfly harness

The butterfly harness provides maximum anterior support, security and freedom of movement for clients who have limited trunk control while sitting.



### Anterior support/tray

The padded anterior support/tray promotes the forward leaning positioning that best facilitates the toileting process.





**Laterals**

Independently adjustable laterals secure clients with limited trunk control. Laterals come in two sizes.

**Small** 5" x 4", **large** 6½" x 5"



**Laterals and chest strap**

With the addition of a wide chest strap to the laterals, clients with low upper torso control gain added trunk support and security.



**Hip guides**

Hip guides reduce the seat width, allowing for years of use as the client grows. They align the torso to aid trunk control and provide spine alignment. They adjust in two increments and can be removed with the armrests for transfer.

**Small** 7" x 3", **large** 11" x 3"



**Pan**

The pan facilitates off-toilet functionality, and its durable plastic construction with a pour spout makes clean-up simple. Its extra depth ensures it will not spill, even when the chair is tilted.

**Small** 2½ quarts, **medium** 4 quarts, **large** 5 quarts



**Bowl adapter**

For situations where the hole in the HTS does not align perfectly with the toilet, the bowl adapter is inserted under the seat pad. It can also be used with the large HTS to create a secure fit over a round front toilet.



## Components (continued)



Splash guard



Standard deflector

### Deflectors

Deflectors keep caregiver and client clean during urination. Options include a low deflector built into the seat pad, a removable soft standard deflector (requires seat pad) and a splash guard that attaches to the seat belt.



### Abductor

By separating the knees and helping to control leg muscles, the abductor prevents scissoring of the legs and promotes good joint and body alignment. It also connects to the standard deflector to maintain cleanliness.

**Small** 3" x 5" x 5", **large** 4" x 5½" x 5½"



### Calf rest and ankle straps

Using the calf rest enhances the client's comfort and sense of security, while ankle straps secure the feet of clients with limited muscle control and excursion movement, enabling a moderate range of movement.



### Footboard for use with mounting bar

This footboard can be attached to the HTS when it is mounted on a toilet with the mounting bar.

**Note:** The footboard should not be used as a step when used with the mounting bar.



### Large casters

The large casters improve mobility by clearing higher thresholds or other obstacles. They add approximately 1" to the height from floor.



## Portability base with carry bag

**The portability kit** gives clients greater independence and their families greater freedom to travel by enabling them to take the HTS on the road. The lightweight folding frame and carry bag (for small and medium sizes) make toileting possible while traveling. Using the HTS eliminates contact with public toilets for better hygiene.

**Note:** The folding base is not intended for use as a primary base.



## Tub base


**The tub base** adds even more versatility to the HTS. The small and medium seat and back attach to the tub base for comfortable showering and hair washing, eliminating the need to purchase a separate bath chair. The tub base tilts the seat back 5°. The reduced footprint accommodates narrow and wide tub bottoms alike.


**Note:** The pan can be used with the tub base.



# HTS dimensions



User dimensions (inches)	Z110 small	Z120 medium	Z130 large
Height	30–46	42–56	50–74
 <p><b>Key user dimension: height</b> The user's overall height is a general guide to help you select the appropriate size of HTS. Choose the model that allows for growth.</p> <p><b>Important:</b> Make sure that seat width, depth and height are adequate for each individual user, and that the user's weight does not exceed the maximum working load.</p>			
Item Dimensions (inches)	Z110 small	Z120 medium	Z130 large
Tilt-in-space mobile base footprint	21 x 31	21 x 31	22½ x 34
Tilt-in-space base length when fully extended (incl. headrest)	43	49¼	57¼
Non-tilt base footprint	20 x 28	20 x 28	21 x 29
Seat (front) to footboard height*	8½–13½	10½–15½	13½–17½
Seat (front) to floor; mobile tilt frame (add 1" for Z181 large casters)	15½–21½	15½–21½	17½–23
Seat (front) to floor; mobile non-tilt frame (add 1" for Z181 large casters)	16–20	17–20½	18–22
Seat (front) to floor; stationary non-tilt frame	13–17	14–17½	15–19
Seat depth (one-inch increments)	8–11	11½–14½	14½–17½
Seat width	8 / 10 / 12	10 / 12 / 14	13 / 15 / 17
Hole size in seat pad	open 4 x 9, closed 3 x 8	open 4½ x 12, closed 3½ x 10	open 6 x 14, closed 4½ x 12
Armrest height	5¼	6½	8
Backrest height above seat	16½	19½	24½
Backrest width	11½	13½	16
Distance between laterals	5¾–11½	7¾–13½	11½–17¼
Tilt in space FORWARD (degrees)	**5°	**5°	**5°
Tilt in space BACK (degrees)	**25°	**25°	**25°
Backrest angle adjustment (degrees)	6° fwd, 6° and 18° back	6° fwd, 6° and 18° back	6° fwd, 6° and 18° back
Max. clear height under seat (tilt frame)	20½	20½	20¾
Max. clear height under seat (stationary non-tilt frame)	15	15	16½
Max. clear height under seat (mobile non-tilt frame)	18	18	19½
Portability base clearance (fits over ADA compliant toilets)	18½	18½	N/A
Bottom of frame to floor; tilt-in-space mobile base	2	2	2
Tub base footprint	23 x 16½ at feet / 19 at top	23 x 16½ at feet / 19 at top	N/A
Tub base height (floor to seat pad)	11 at front, 9½ at back	11 at front, 9½ at back	N/A
Max. working load (lbs)	75	150	***250

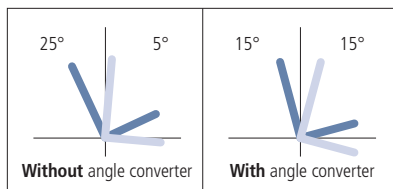

\*When the mobile base is in the lower positions and the footboard is fully extended, it may prevent the front casters from swiveling. In this case, either change the footboard angle or raise the seat height to give the casters room to swivel.

\*\*To achieve 15° of forward tilt for ease of transfer, order the **tilt angle converter**, Z169 \$55. (The range of adjustment will be 15° back and 15° forward. See reference charts at left.)

\*\*\*Max. working load of the large footboard is 150 lbs.

# 2020 order form

Use dimension chart to select appropriate size.

		Z110 small	Z120 medium	Z130 large
<b>* Seat and back</b> (includes seat belt)		Z111 \$395 <input type="checkbox"/>	Z121 \$465 <input type="checkbox"/>	Z131 \$625 <input type="checkbox"/>
<b>* Base</b>	Mobile (tilt-in-space with footboard)	Z112 \$690 <input type="checkbox"/>	Z122 \$735 <input type="checkbox"/>	Z132 \$915 <input type="checkbox"/>
	Mobile (non-tilt with footboard)	Z113 \$450 <input type="checkbox"/>	Z123 \$490 <input type="checkbox"/>	Z133 \$680 <input type="checkbox"/>
	Stationary (non-tilt with footboard)	Z184 \$475 <input type="checkbox"/>	Z185 \$510 <input type="checkbox"/>	Z186 \$770 <input type="checkbox"/>
	Tub base (non-tilt)	Z138 \$205 <input type="checkbox"/>	Z138 \$205 <input type="checkbox"/>	N/A
	Mounting bar (round-front toilet)	Z115 \$80 <input type="checkbox"/>	Z115 \$80 <input type="checkbox"/>	Z135 \$100 <input type="checkbox"/>
	Mounting bar (elongated toilet)	Z125 \$80 <input type="checkbox"/>	Z125 \$80 <input type="checkbox"/>	Z135 \$100 <input type="checkbox"/>
<b>* Armrests</b> (pair)	CHOOSE Low	Z101 \$60 <input type="checkbox"/>	N/A	N/A
	CHOOSE Medium	Z102 \$65 <input type="checkbox"/>	Z102 \$65 <input type="checkbox"/>	Z102 \$65 <input type="checkbox"/>
	CHOOSE High	N/A	Z103 \$75 <input type="checkbox"/>	Z103 \$75 <input type="checkbox"/>
<b>Casters for mobile base</b>	CHOOSE Small casters (3")	Z195 \$175 <input type="checkbox"/>	Z195 \$175 <input type="checkbox"/>	Z195 \$175 <input type="checkbox"/>
	CHOOSE Large casters (4")	Z181 \$250 <input type="checkbox"/>	Z181 \$250 <input type="checkbox"/>	Z181 \$250 <input type="checkbox"/>
<b>Seat and back pads</b> (recommended)	CHOOSE Rear-open seat pad	Z153 \$175 <input type="checkbox"/>	Z163 \$220 <input type="checkbox"/>	Z173 \$285 <input type="checkbox"/>
	CHOOSE Rear-closed seat pad	Z154 \$175 <input type="checkbox"/>	Z164 \$220 <input type="checkbox"/>	Z174 \$285 <input type="checkbox"/>
	CHOOSE No-hole seat pad	Z187 \$175 <input type="checkbox"/>	Z188 \$220 <input type="checkbox"/>	Z189 \$285 <input type="checkbox"/>
<b>Headrest</b>		Z104 \$200 <input type="checkbox"/>	Z104 \$200 <input type="checkbox"/>	Z104 \$200 <input type="checkbox"/>
<b>Push handles</b> (pair)		Z199 \$110 <input type="checkbox"/>	Z199 \$110 <input type="checkbox"/>	Z199 \$110 <input type="checkbox"/>
<b>Butterfly harness</b>		Z116 \$105 <input type="checkbox"/>	Z126 \$115 <input type="checkbox"/>	Z136 \$140 <input type="checkbox"/>
<b>Anterior support/tray</b>		Z145 \$190 <input type="checkbox"/>	Z155 \$235 <input type="checkbox"/>	Z165 \$310 <input type="checkbox"/>
<b>Laterals</b> (pair)	CHOOSE Small	Z105 \$190 <input type="checkbox"/>	Z105 \$190 <input type="checkbox"/>	Z105 \$190 <input type="checkbox"/>
	CHOOSE Large	N/A	Z106 \$210 <input type="checkbox"/>	Z106 \$210 <input type="checkbox"/>
	CHOOSE Small with strap	Z107 \$235 <input type="checkbox"/>	Z107 \$235 <input type="checkbox"/>	Z107 \$235 <input type="checkbox"/>
	CHOOSE Large with strap	N/A	Z108 \$240 <input type="checkbox"/>	Z108 \$240 <input type="checkbox"/>
<b>Hip guides</b> (pair)	CHOOSE Small	Z109 \$115 <input type="checkbox"/>	Z109 \$115 <input type="checkbox"/>	Z109 \$115 <input type="checkbox"/>
	CHOOSE Large	N/A	Z149 \$130 <input type="checkbox"/>	Z149 \$130 <input type="checkbox"/>
<b>Pan</b>		Z117 \$50 <input type="checkbox"/>	Z127 \$60 <input type="checkbox"/>	Z137 \$70 <input type="checkbox"/>
<b>Bowl adapter</b>		Z196 \$50 <input type="checkbox"/>	Z196 \$50 <input type="checkbox"/>	Z197 \$65 <input type="checkbox"/>
<b>Deflector</b>	CHOOSE Standard (requires seat pad)	Z118 \$45 <input type="checkbox"/>	Z118 \$45 <input type="checkbox"/>	Z128 \$60 <input type="checkbox"/>
	CHOOSE Splash guard	Z119 \$50 <input type="checkbox"/>	Z119 \$50 <input type="checkbox"/>	Z129 \$65 <input type="checkbox"/>
<b>Abductor</b>	CHOOSE Small	Z124 \$100 <input type="checkbox"/>	Z124 \$100 <input type="checkbox"/>	N/A
	CHOOSE Large	N/A	Z144 \$120 <input type="checkbox"/>	Z144 \$120 <input type="checkbox"/>
<b>Ankle straps</b> (pair, requires footboard)		Z156 \$75 <input type="checkbox"/>	Z156 \$75 <input type="checkbox"/>	Z157 \$80 <input type="checkbox"/>
<b>Calf rest</b> (requires footboard)		Z158 \$65 <input type="checkbox"/>	Z158 \$65 <input type="checkbox"/>	Z159 \$70 <input type="checkbox"/>
<b>Footboard</b> (for mounting bar attachment)		Z146 \$175 <input type="checkbox"/>	Z147 \$220 <input type="checkbox"/>	Z148 \$275 <input type="checkbox"/>
<b>Portability base with carry bag</b>		Z139 \$495 <input type="checkbox"/>	Z139 \$495 <input type="checkbox"/>	N/A
<b>Total</b>		<b>Z110</b> \$ _____	<b>Z120</b> \$ _____	<b>Z130</b> \$ _____

\* Required